



REHAU Face Shield PET-G


Safe and easy to use

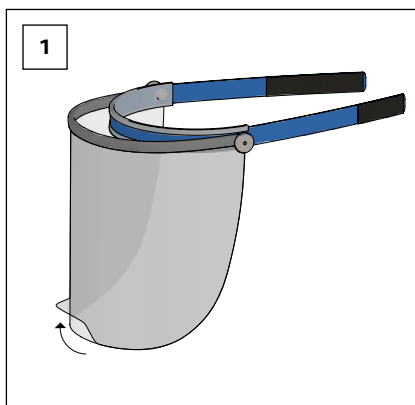

Remove protective foil as illustrated.

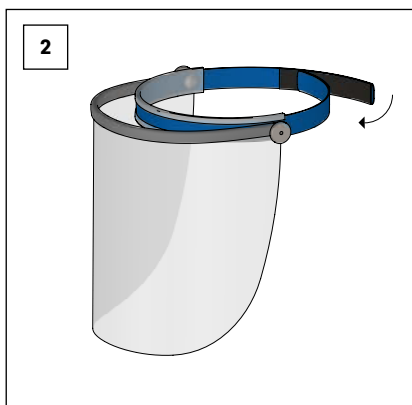

Put Face Shield on the head and fasten with velcro close tightly.

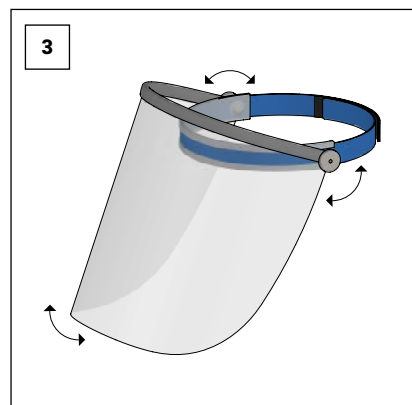

Slightly loosen the fixing screws, adjust the angle of the umbrella and pull the fixing screws tight again.



Safety advice 1

The REHAU Face Shield PET-G protects against dripping and splashing liquids and thus reduces the risk of droplet infection. A complete protection against viruses, bacteria and other biological agents is, due to its design, limited. It can be combined with face masks and goggles. The visor has optical class 1.

Cleaning instructions



Use pure soap suds for cleaning!

Disinfection with standard household disinfectants.



1. apply soapy water with a soft towel
2. rinse with clean water
3. rub dry with a soft towel

- The REHAU Face Shield PET-G fulfils the basic health and safety requirements of the PSA Regulation 2016/425 and EN 166:2001 (the REHAU Face Shield PET-G offers no protection against mechanical effects e.g. high speed particle).
- The face shield must be replaced after use. Reuse is at the option of the user.
- The face protection can be combined with face masks, goggles and head covers.

REHAU Face Shield PET-G

Marking

Frame: REH 166 3 S CE

REH = manufacturer code

166 = test standard EN 166

3 = resistance to splash of liquids

S = material code for improved strength properties

CE = European Conformity

Vision panel: REH 1 S CE

REH = manufacturer code

1 = optical class

S = material code for improved strength properties

CE = European Conformity

- If face shields are not used, they should be largely protected from light and kept dry in smoke-tight rooms. When storing, make sure that the visors are not touching the ground to avoid scratches.
- In case of discolored, strongly scratched visors or if visors containing stuck particles, the face shields must be replaced. The visors are available individually as spare parts. This also applies if the face shields are otherwise damaged, or if the fixation screws can no longer be tightened sufficiently to allow safe and secure positioning of the visor.

Accessories:

The visors are available individually as spare parts.

Mat no.: 13365191001

Expiration date at dry, light protected storage:

If stored properly in a dry place without direct sunlight, the Face Shield will retain its properties for at least 5 years.

Harmonisation rules:

Regulation 2016/425/EU

Directive 2001/95/EC

EN 166:2001

Risk assessment:

→ see safety instruction 1

- offers no protection against types of radiation (e.g. solar radiation, UV radiation, welding, IR)
- is not fire resistant
- a damaged face shield can affect visibility and protection
- Materials that come into contact with the skin can cause an allergic reaction in sensitive persons.

The notified body DIN CERTCO Gesellschaft für Konformitätsbewertung mbH, Alboinstr. 56, D-12103 Berlin, notified body no. 0196 has issued the type examination certificate C6793REH/R0.

The EU Declaration of Conformity can be found at:

www.rehau.com/faceshield

© REHAU AG + Co
Rheniumhaus
Otto-Hahn-Str. 2
95111 Rehau
Germany